



Pro bezpe nost vašich
d tí

A gyermekek biztonsága
érdekében

Zum Schutz Ihrer Kinder.

Per la sicurezza dei bambini:

Af hensyn til dine børns sikkerhed

Voor de veiligheid van uw
kinderen.

Para la seguridad de sus hijos

For dine barns sikkerhet

Lastesi turvallisuuden vuoksi

Bezpiecze stwo dzieci

Pour la sécurité de vos enfants

Para a segurança das crianças

For the safety of your children

För barnens säkerhet

Bellman[®]audio
FOR EASY COMMUNICATION[™]



Bellman & Symfon[®]

Bellman & Symfon AB
Södra Långebergsgatan 30
SE-421 32 Västra Frölunda Sweden
Tel. +46 31 68 28 20 Fax. +46 31 68 28 90
info@bellman.com www.bellman.com



BE2020_005MAN014





Bellman Audio Maxi

Digital communication aid

MODEL: BE2020

- APP
- SE
- PT
- PL
- NO
- NL
- IT
- HU
- GR
- FR
- FI
- ES
- EN
- DK
- DE
- CZ



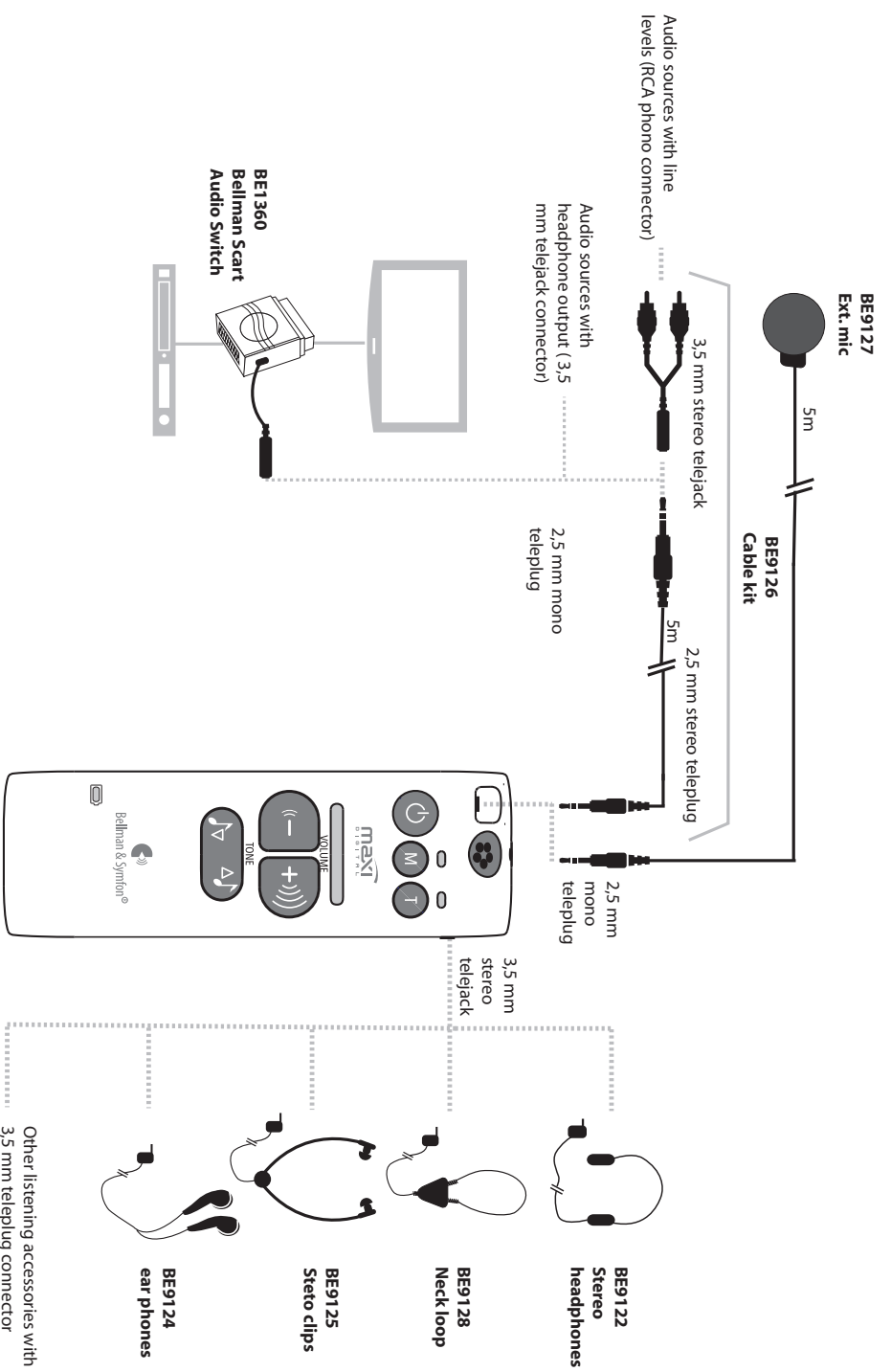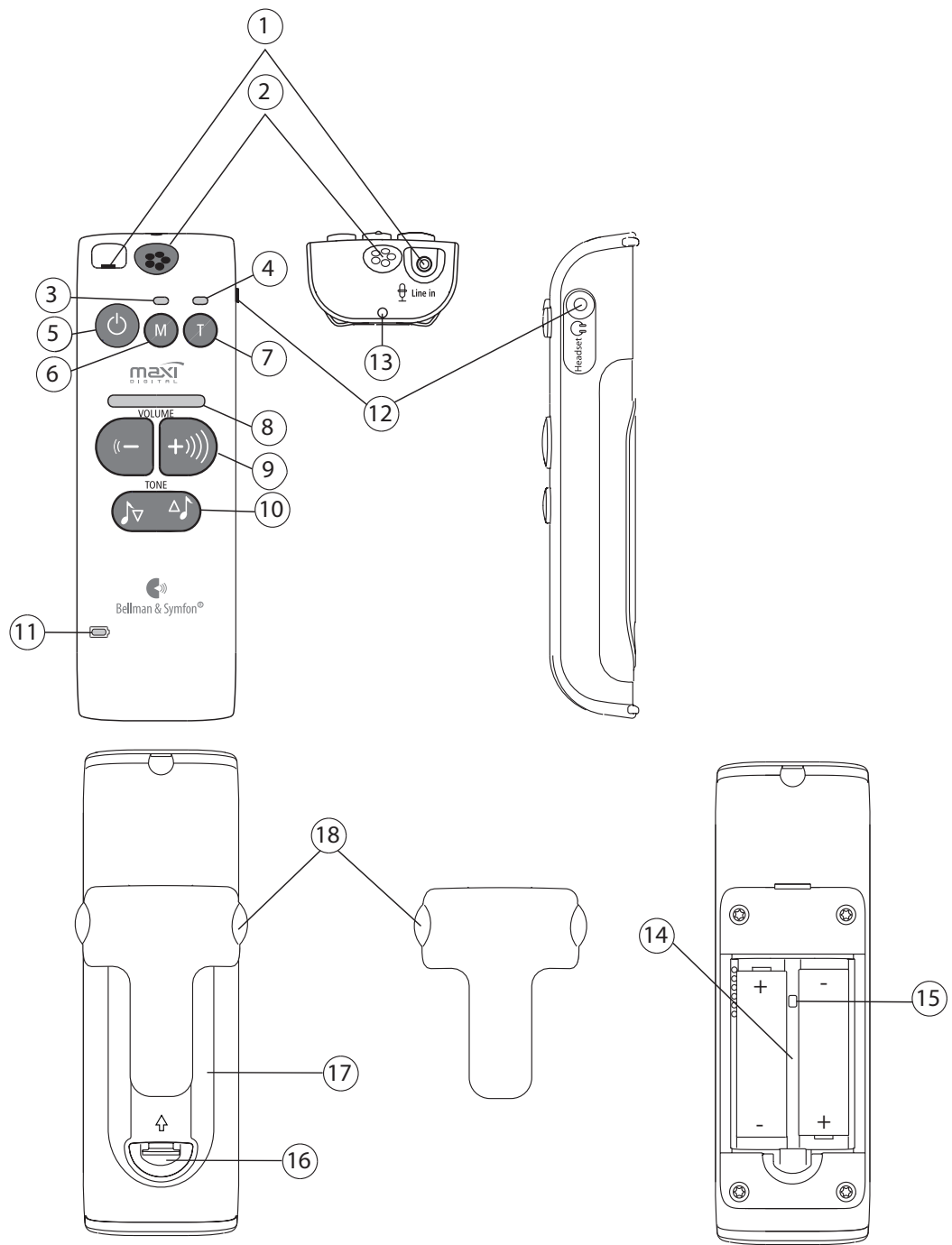

Bellman® Audio Maxi Digital Communication Aid, BE2020

Thank you for choosing products from Bellman & Symfon.

Bellman Audio is an assistive listening system. The system consists of a number of units which facilitate listening under different circumstances. Bellman Audio products make everyday life easier for many people and give them the freedom to freely pursue their interests.

Read through the entire user manual first before starting to use the product.

Refer to the diagram showing the accessories and connection options on the inside of the cover.

Getting started

Unpacking, installing and testing the unit

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

Warning!

Function

Built-in microphone

Telecoil



Selecting the signal source

(M)




(T)

External sound source

Combined signal sources

(M)

Clip

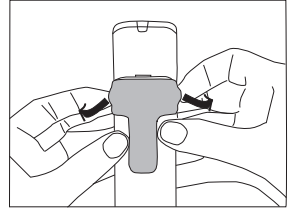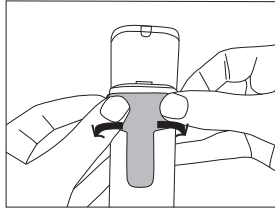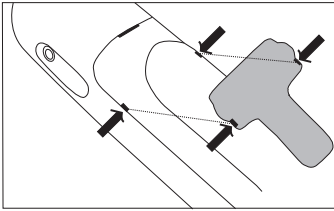

Place the clip against the battery cover and press firmly so that it locks on.

Remove the clip by using the small round handles and pressing out from the sides.

Indicators and settings

Volume and Tone LED Indicator

Low battery LED

Volume control



Tone control



Troubleshooting in brief

Symptom	Solution

